AXIOM

BACKS

PB | PL | LB | DL

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TABLE OF CONTENTS

	IMPORTANT CONSUMER INFORMATION.....	2
	Axiom Back Features.....	2
A.	Intended Use.....	3
B.	Selecting the Proper Seating Product.....	3
C.	Become Familiar with this Manual.....	3
D.	Warnings.....	3
E.	Symbol Glossary	3
F.	Check Your Skin Daily	4
G.	General Warnings.....	4
H.	Protecting Your Axiom Back.....	5
I.	Cover Orientation	5
	Set-Up, Adjustment and Care.....	6
A.	Fitting Your Axiom Back and Wheelchair	6
B.	Pushing or Lifting	10
C.	Axiom Back Transit Safety.....	11
D.	Maintenance	11
E.	Cleaning.....	12
F.	Reassembling Your Axiom Back After Cleaning.....	13
	KI MOBILITY LIMITED WARRANTY.....	14

Axiom Back

IMPORTANT CONSUMER INFORMATION


NOTICE: This manual contains instructions for use with the Ki Mobility Back products. This information must be provided to the consumer of this product. Do not remove this manual before delivery to the consumer of this product.

Supplier: This manual must be given to the consumer of this product.

Consumer: Read the entire owner's manual prior to using this product and keep on-hand for future reference.

NOTICE: Check all parts for shipping damage. In case of damage, DO NOT use. Contact the supplier or Ki Mobility for further instructions.

Axiom Back Features

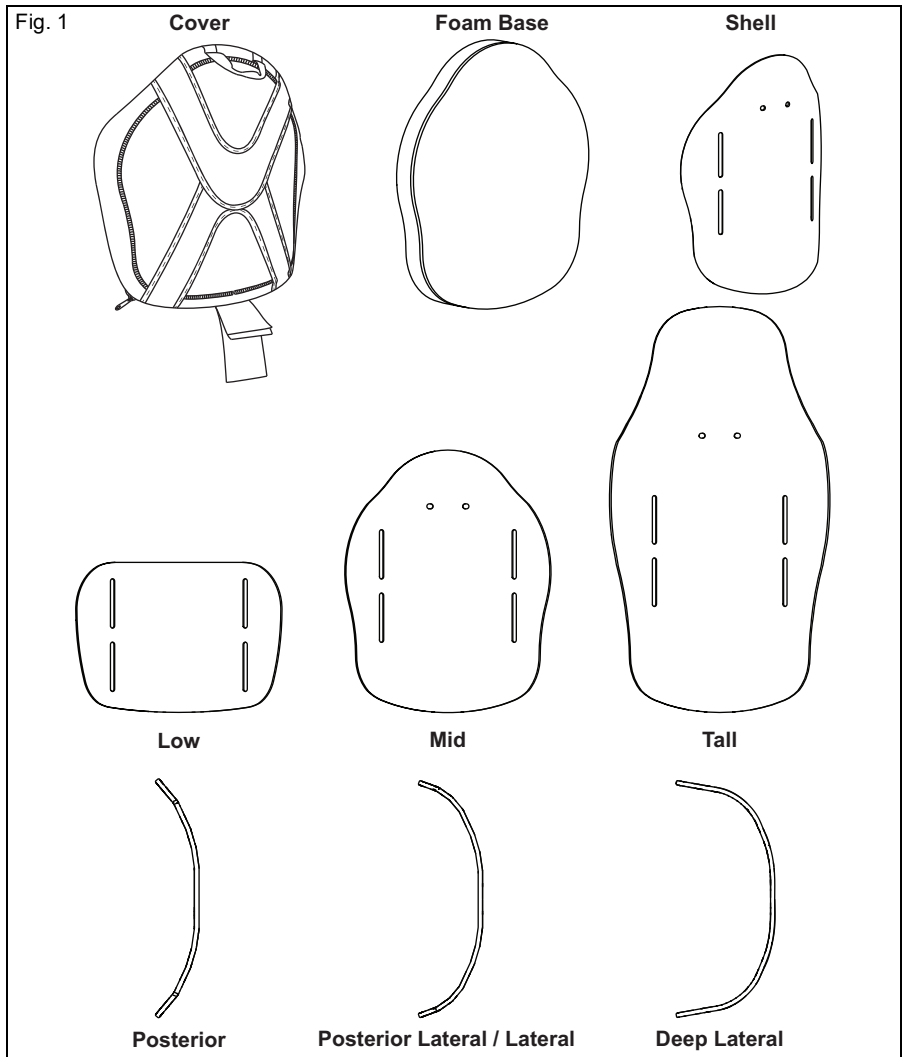A. Intended Use

Ki Mobility Wheelchair Backs are intended to provide postural support, pressure distribution and improved sitting tolerance.

B. Selecting the Proper Seating Product

Ki Mobility recommends that you consult with a licensed clinician (i.e. Physician or therapist) trained in wheelchair seating and positioning before selecting any seating and positioning product. This will help ensure you receive the right product for your specific needs. Axiom Seating and Positioning products should only be installed by an authorized Ki Mobility supplier.

C. Become Familiar with this Manual

Before using this Axiom Back, you and each person who may assist you, should read this entire owner's manual and follow all instructions. Review the Warnings often, until they are second nature to you.

D. Warnings

The word "WARNING" refers to a hazard or unsafe practice that may cause severe injury or death to you or to other persons.

E. Symbol Glossary

SYMBOL	DEFINITION
	Indicates the medical device manufacturer.
	Date of manufacture (YEAR/MM/DD).
	Indicates the need for the user to consult an instruction for safety reasons such as cautions and warnings. If presented on the medical device or packaging, it indicates the need for the user to consult the instructions for safety reasons such as cautions and warnings.
	Indicates the item is a medical device.
	Indicates the manufacturer's serial number so that a specific medical device can be identified.
	Indicates the authorized representative in the European Community.
	Indicates the manufacturer's declaration that the product meets the requirements of the applicable EC directives.
	Indicates the need for the user to consult the instructions for use.
	Indicates the need for the user to consult the listed website for instructions for use in an electronic format.
	Indicates a website where a user may obtain additional information about the medical product.
	Indicates a specified maximum weight limit (lbs/kg).
	Indicates a transit securement point.
	Indicates a potential pinch point.

F. Check Your Skin Daily

Axiom Seating and Positioning products are designed to improve pressure distribution and increase sitting tolerance. Whether the Axiom Back provided will meet your individual needs cannot be assured in advance. The integrity of your skin can be affected by many aspects of your daily life and medical condition, including the use of this product. Be sure to follow any skin care regimens established by your clinician.

1. Users of this product should make sure their skin is inspected daily for changes. If you see changes, discontinue the use of this product immediately and consult with your clinician. A key clinical indicator of tissue breakdown is skin redness. If your skin develops redness, discontinue the use of this product immediately and consult your clinician.

WARNING: Failure to properly select an Axiom Back that fits your wheelchair can affect your skin integrity which can result in serious injury or death.

WARNING: Failure to perform proper skin inspection or continuing to use an Axiom Back after skin changes are present can affect your skin integrity, resulting in serious injury or death.

G. General Warnings

WARNING: Do not use or install this Axiom Back without first reading and understanding this manual. Please contact your authorized Ki Mobility supplier or your clinician if you are unable to understand the Warnings and Instructions. Failure to do so could result in serious injury or death.

WARNING: The installation of an Axiom Back on a wheelchair could affect the center of gravity of the wheelchair. Changes in your center of gravity may affect your stability in your wheelchair, resulting in tipping over or falling from your wheelchair which may result in serious injury. Always review the instructions for use of your wheelchair to see if changes to the wheelchair may be needed to provide sufficient stability after adding an Axiom Back.

WARNING: The integrity of your skin can be affected by many aspects of your daily life and medical condition, including the use of this product. Be sure to follow any skin care regimens established by your clinician. Consumers of this product should make sure their skin is inspected routinely for changes as directed by their clinician. Failure to do so could result in serious injury or death.

H. Protecting Your Axiom Back

Avoid prolonged exposure to adverse environmental conditions. This may include excessive exposure to sunlight and temperatures, either hot or cold. This can result in the degradation of the performance of your Axiom Back.

WARNING

WARNING: Failure to protect your Axiom Back may lead to changes that can affect your skin integrity, resulting in serious injury or death.

Although your Axiom Back has been manufactured in accordance with its labeled flammability standards, you should avoid exposing your cushion to open flames or extreme heat sources such as a burning cigarette. This may cause your cushion to ignite or emit harmful gases.


WARNING

WARNING: Failure to protect your Axiom Back from extreme heat may cause your Axiom Back to ignite or emit harmful gases, resulting in serious injury or death.

I. Cover Orientation

Your Axiom Back was designed to be seated in a specific orientation and it must be placed properly in your wheelchair with the cover oriented correctly with the foam base. It is labeled to indicate the proper orientation. See Fig. 2.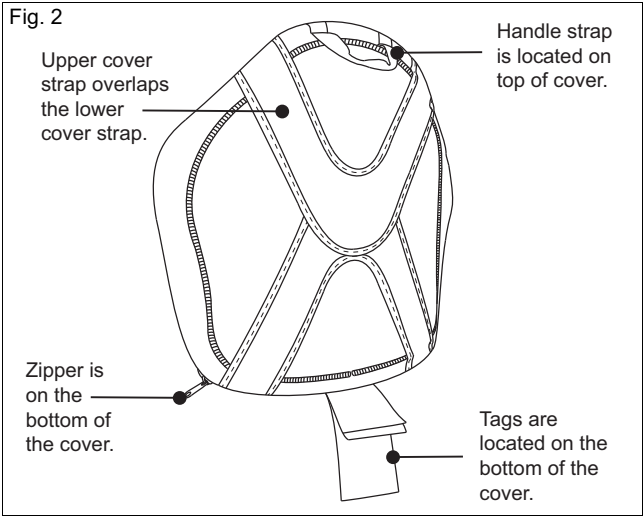

WARNING

WARNING: Failure to properly orient the Axiom Back cushion and cover can affect your skin integrity, resulting in serious injury or death.

Set-Up, Adjustment and Care

A. Fitting Your Axiom Back and Wheelchair

WARNING

WARNING: The user of this equipment is responsible for making sure that it has been set up and adjusted by an Authorized Ki Mobility Supplier under the advice of a healthcare professional. The Axiom Back and hardware may require periodic safety and function checks. Adjustments should be performed by your Authorized Ki Mobility Supplier. Always use parts and/or accessories that have been recommended or approved by Ki Mobility.


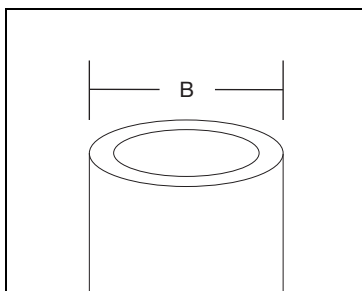
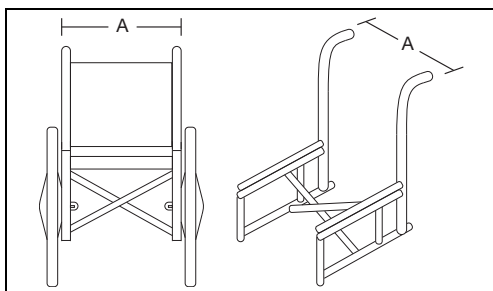
TABLE 1

Axiom Back Width (all contours)	Width of back canes from outside to outside of the tubes (A)			
	Minimum Width		Maximum Width	
	Inch	Cm	Inch	Cm
12"	12	30.5	14	35.6
14"	14	35.6	16	40.6
16"	16	40.6	18	45.7
18"	18	45.7	20	50.8
20"	20	50.8	22	55.9
22"	22	55.9	24	61.0


TABLE 2

Axiom Back Width (all contours)	Cane Diameter (B)			
	Minimum Width		Maximum Width	
	Inch	Cm	Inch	Cm
12" - 26"	3/4	1.9	1 1/8	2.85

NOTE: Backrest cane clamp bolts come in two different sizes.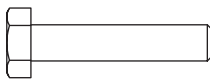

30mm (Used for 3/4" and 7/8" back canes)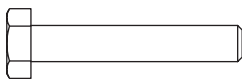


35mm (Used for 1" and 1-1/8" back canes)

A. Fitting Your Axiom Back and Wheelchair (Continued)


General Setup

NOTICE: If ordered with installation on chair, skip to Step 4.


1. Remove backrest shell from cover. Install backrest bracket to shell with stud plate (Fig. 3:A), two washers (Fig. 3:B) and two dome nuts (Fig. 3:C). Install so stud plate studs are 1" from the top of each bracket slot. Loosely tighten hardware. Repeat on opposite side.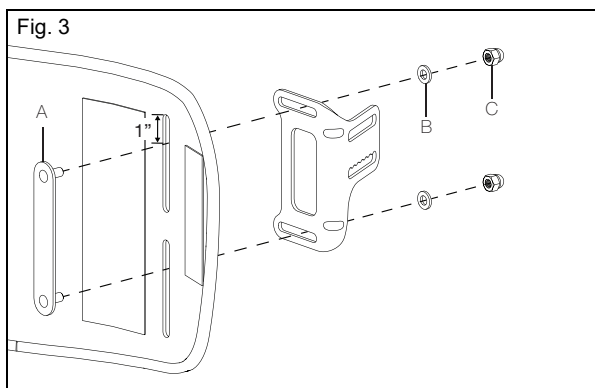

2. Install housing (Fig. 4:C) to bracket with two bolts (Fig. 4:A), two lock washers (Fig. 4:B), two washers (Fig. 4:D), two spring discs (E) and a nut plate (Fig. 4:F). Loosely tighten hardware and repeat on opposite side.

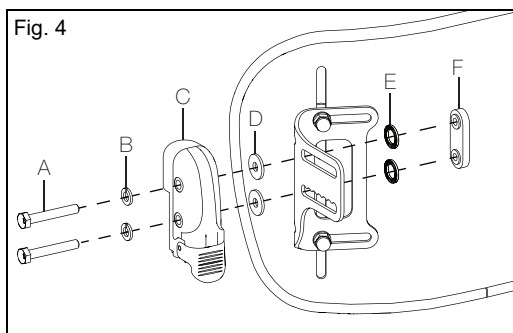
3. Install backrest mount clamp (Fig. 5:B) to mount plate (Fig. 5:C) using two lock washers (Fig. 5:E), two spherical washers (Fig. 5:D) and a dual nut (Fig. 5:A). Bolts should thread inwards from the outside of the chair. Choose bolts, from the two sizes provided, that adequately thread through dual nut without coming up short or sticking out excessively. Loosely tighten hardware. Repeat with second set of backrest mount clamps.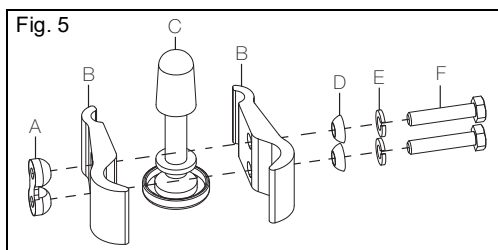


A. Fitting Your Axiom Back and Wheelchair (Continued)

4. Install the backrest assembly onto the backrest mounts. See Fig. 6.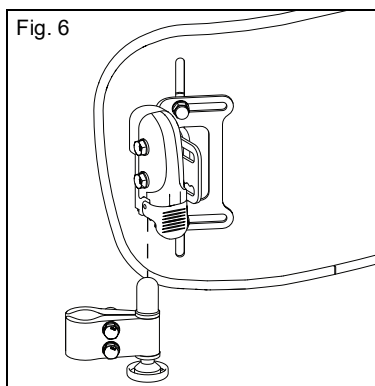

5. Visualize the user seated on the chair with a cushion and where the clamps will mount on the backrest canes. Ensure there will be no interference with the backrest canes. The bottom edge of backrest will typically fall between an 1" to 2" above the top of the cushion. Loosen the mount clamps and install onto backrest canes. See Fig. 7.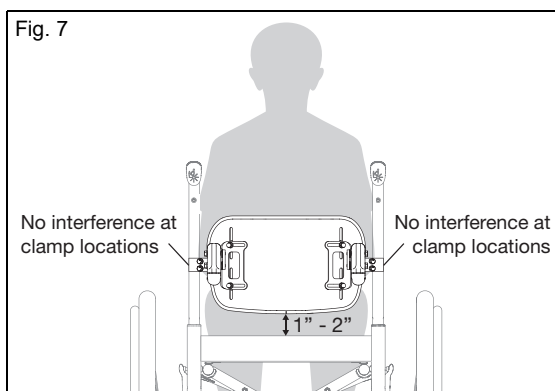


6. Adjust backrest so it appears centered between the two back canes and both clamps are set at the same height on both sides. Using a tape measure is recommended to ensure both sides are set up equally. Once final position is achieved with only bolt tightened on each hardware surface, then release and test hardware latch and release. Tip: Only loosen and snug up one bolt at a time until desired position is achieved.
7. Final latch should activate when pulling rearward on the very top part of the back. If latch doesn't click into place, loosen the side that isn't engaging and manually manipulate the latch to engage. Repeat step 7 if necessary.
8. Tighten all hardware enough so pieces are secured during travel or until user specific adjustments are made later. Overtightening the back post clamp hardware may cause damage to backrest canes.
9. Reinstall backrest cover. Remove and reinstall backrest multiple times to ensure alignment is correct and all pieces are secured in place.


A. Fitting Your Axiom Back and Wheelchair (Continued)

Custom Adjustments

1. Adjust the height of the backrest, if needed, by loosening the two nuts (Fig.8:A) on the backrest bracket. Slide shell up or down to achieve desired height and retighten hardware. Ensure both sides are set the same.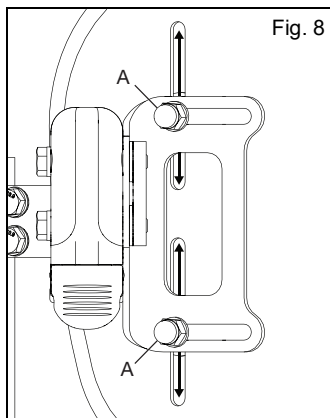

2. The angle and the depth of the backrest are adjusted, if needed, at the same time. Loosen the two bolts (Fig. 9:A) on the bracket and adjust the angle and depth of the backrest in one movement. Retighten hardware once desired configuration is achieved.

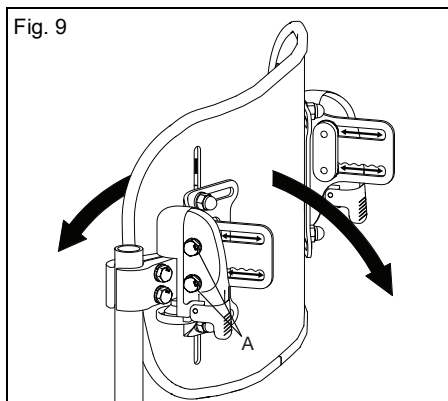3. Tighten all hardware to 110 in-lbs except the acorn nuts which are tightened to 80 in-lbs. Overtightening the back post clamp hardware may cause damage to backrest canes.
4. With no user in chair, remove and reinstall backrest multiple times to ensure alignment is correct and all pieces are secured in place.

WARNING: Failure to properly orient the Axiom Back and cover can affect your skin integrity which can result in serious injury or death.

WARNING: Failure to properly select the Axiom Back and cover that can be retained in the wheelchair can cause you to fall from your wheelchair, resulting in serious injury or death.

B. Pushing or Lifting

WARNING: Do not use the Axiom Back to lift or push the wheelchair.

1. Do not use the Axiom Back handle to lift the weight of the wheelchair. It is intended only for Axiom Back removal.
2. Pushing or pulling is considered improper use of the Axiom Back and may cause the Axiom Back to unexpectedly detach from the wheelchair. Failure to heed these warnings can cause damage to your chair, or cause a fall, tip-over or loss of control may occur and cause severe injury to the user or others.

C. Axiom Back Transit Safety

The Axiom Back has been tested according to the dynamic test requirements of ANSI/RESNA WC-4, Section 20: Wheelchair seating systems for use in motor vehicles or ISO 16840-4 Wheelchair seating - Part 4: Seating systems for use in motor vehicles. A complete system of wheelchair frame, seating, wheelchair tie-down and occupant restraint systems and a properly equipped motor vehicle, that have all complied with the standards mentioned in this section, should be in place before using an Axiom Back in a wheelchair that is used as a seat in a motor vehicle.

WARNING: Failure to follow all instructions for installation, use and maintenance within this manual as well as the transit instructions listed below, could result in serious injury or death when using the Axiom Back in a wheelchair that is used as a seat in a motor vehicle.

1. If possible and feasible, the rider should transfer to the Original Equipment Manufacturer vehicle seat and use the vehicle restraints.
2. The distance between the top of the user's shoulder and the top of the Axiom Back should not exceed 5.5" (14 cm).
3. The wheelchair must be labeled as appropriate for use as a seat in a motor vehicle, dynamically tested to the performance requirements of ISO Standard 7176-19 and installed, used and maintained according to the manufacturer's instructions.
4. If the original wheelchair seat was replaced, the new seat must be approved for wheelchair transit, installed and used as indicated in the manufacturer's instructions.
5. When using your wheelchair as a seat in a motor vehicle the rider must be in a forward-facing position.
6. The rider must use a Wheelchair Tie-down and Occupant Restraint System that complies with ANSI-RESNA WC-4, Section 18: Wheelchair tie-down and occupant restraint systems for use in motor vehicles or ISO 10542-1 Technical systems and aids for disabled or handicapped persons -- Wheelchair tie-down and occupant-restraint systems -- Part 1: Requirements and test methods for all systems and installed, used and maintained according to the manufacturer's instructions.
7. Use of lap belts, chest straps, shoulder harnesses, any other positioning strap system or positioning accessory should not be used, or relied on as an occupant restraint, unless it is marked as such by the manufacturer in accordance with ANSI-RESNA WC-4, Section 18 or ISO 10542-1, Part 1.

C. Axiom Back Transit Safety (Continued)

8. Use of headrests, lateral supports or other positioning accessories should not be relied on as an occupant restraint, unless it is marked as such by the manufacturer in accordance with RESNA WC-4, Section 18 or ISO 10542-1, Part 1 or ANSI/RESNA WC-4, Section 20: Wheelchair seating systems for use in motor vehicles or ISO 16840-4 Wheelchair seating -- Part 4: Seating systems for use in motor vehicles.
9. Aftermarket accessories such as trays, oxygen tank holders, oxygen tanks, IV poles, backpacks, pouches and other items should be removed and secured separately in the motor vehicle. In the event of an accident, these items can become dangerous projectiles which may result in serious injury or death for you or other occupants of the motor vehicle.
10. This product is intended for use by adults ranging in weight from 79 to 300 lbs (36 to 136 kg).
11. If the Axiom Back has been involved in an accident, you should not continue to use it, as it may have suffered fatigue that may not be visible.

D. Maintenance

Regular cleaning and maintenance may help extend the life of your Axiom Back. You should inspect your Axiom Back monthly. During cleaning, component inspection is recommended.

1. Check Cover for Tears and Excessive Wear
 - a. Remove cover and check for abnormalities (inside and out).
 - b. Check the foam for breakdown or degradation.
2. Check the hardware for any looseness, signs of stripping, degradation or deformities.

WARNING: If you find any abnormalities, such as the foam breaking down or damage to the hardware or fasteners, discontinue the use of this product immediately and consult with your Authorized Ki Mobility Supplier. Failure to do so can affect your skin integrity, resulting in serious injury or death.

WARNING: The owner of this equipment is responsible for making sure that it has been set up and adjusted by an Authorized Ki Mobility Supplier under the direction of a healthcare professional. It may require periodic safety and function checks or certain in-use adjustments that can be performed by the user, caregiver or Authorized Ki Mobility Supplier.

1. Fasteners: Ki Mobility recommends that all fasteners be inspected for proper function, such as loose bolts or broken components every six months. Loose fasteners should be retightened according to the installation instructions.
2. Replacing Components: Contact your Authorized Ki Mobility Supplier immediately to replace any components. Discontinue use of the system after identifying loose or broken components.

E. Cleaning


1. Axiom Back Cover

- a. Remove the cover from Axiom Back by detaching the Velcro®. Unzip the zipper on the bottom of the cover.
- b. Slightly roll and remove the foam to turn the cover inside out.
- c. Zip the cover closed to avoid snagging.
- d. Machine wash in hot water (40°C).
- e. Drip and tumble dry on No Heat.

NOTICE: Do not dry clean covers or use industrial washers or dryer to clean covers. Do not steam autoclave. Do not bleach. Laundry care symbol definitions are shown in Fig. 11.

2. Axiom Back Assembly

- a. Remove the whole back assembly to make cleaning easier. See Fig. 10.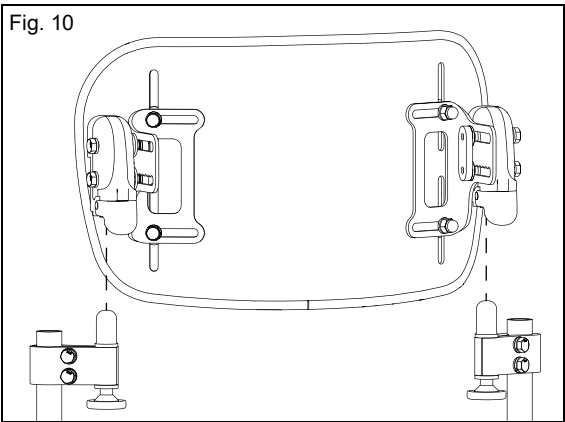

- b. The Axiom Back Shell should be wiped clean with soap and water on a regular basis to prevent buildup of dirt and grime.


3. Positioning Components

- a. Positioning components can be wiped clean with soap and water.
- b. DO NOT immerse positioning components underwater.

Fig. 11 Care & Use Instructions	
	Machine Wash, Hot
	Do Not Bleach
	Tumble Dry, Normal, No Heat
	Do Not Iron
	Do Not Dry Clean

F. Reassembling Your Axiom Back After Cleaning

1. After Cleaning
 - a. Reinstall foam and cover. Slightly roll the foam when installing cover.
2. Back Assembly Reinstallation
 - a. To reattach the Axiom Back and hardware receivers, line up the pins to the receivers.
 - b. Push the Axiom Back into place on the receivers.
 - b. If properly inserted, the pins will "click" into a locked position.

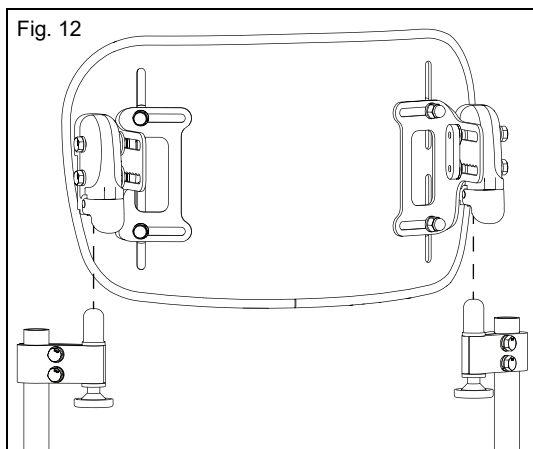WARNING

WARNING: If you are uncertain that you have heard the locking "click", pull up on the Axiom Back using the handle to test and ensure it has been securely locked. If you fail to heed this warning, damage to your equipment, a fall, or loss of control may occur and cause severe injury to the rider or others.

KI MOBILITY LIMITED WARRANTY

The expected life of the product is 5 years.

A. For 24 Months

Ki Mobility warrants the Axiom Back to be free from defects in materials and workmanship for a period of 24 months. This limited warranty applies to:

1. Axiom Back shell and foam

For a period of 24 months (2 years) from the date of purchase, provided normal use, should a defect in materials or workmanship occur, Ki Mobility will, at its option, repair or replace it without charge. This warranty does not apply to punctures, tears, or burns.

B. For 6 Months

Ki Mobility warrants the cushion to be free from defects in materials and workmanship for a period of 6 months. This limited warranty applies to:

1. Removable covers

For a period of 6 months from the date of purchase, provided normal use, should a defect in materials or workmanship occur, Ki Mobility will, at its option, repair or replace it without charge. This warranty does not apply to punctures, tears or burns.

C. Limitations

1. We do not warrant:
 - a. Damage from neglect, accident, misuse, or from improper installation or repair.
 - b. Products modified without Ki Mobility's express written consent.
 - c. Damage from exceeding the weight limit.
2. This warranty is VOID if the original Axiom Back tag is removed or altered.
3. This warranty applies in the USA only. Check with your supplier to find out if international warranties apply.
4. This warranty is not transferable and only applies to the first consumer purchase of this product through an authorized Ki Mobility supplier.

D. What We Will Do

Our sole liability is to repair or replace covered parts. This is the exclusive remedy for consequential damages.

E. What You Must Do

1. Obtain from us, while this warranty is in effect, prior approval for return or repair of covered parts.
2. Claims and repairs should be processed through the nearest authorized supplier. Except for express warranties made herein, all other warranties, including implied warranties of merchantability and warranties of fitness for a particular purpose are excluded.

F. Notice to Consumer

1. If allowed by law, this warranty is in place of any other warranty (written or oral, express or implied, including a warranty of merchantability, or fitness for a particular purpose).
2. This warranty gives you certain legal rights. You may also have other rights that vary from state to state.

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